

# Chair-Check® II Models 72010 & 72011



## Portable, Versatile Fall Monitoring



Bed



Chair



Wheelchair



Toilet



**Rugged design** and the ability to monitor almost any seated or reclined application make the Chair-Check II a universally adaptable fall risk monitoring system. If you must choose just one system for bed, chair, and toilet use, choose Chair-Check II.

- Monitors chair, toilet, wheelchair, or bed, compatible with any Bed-Check® Sensormat® pad.
- Total portability: common 9V alkaline battery provides long life; control unit provides a low battery alert. Secured battery containment discourages tampering. "Go green" with an optional rechargeable battery and charger/AC adapter (Model 72011).
- No on-off switch: monitors automatically with weight on the Sensormat pad to help you meet The Joint Commission requirements.
- Non-latching alarm lets monitoring automatically restart when individual's weight is back on the Sensormat pad, and does not require caregiver intervention.
- Customize Chair-Check II settings to the individual's needs with 3 selectable alarm tones, 2 volume settings, and 0-5 second delay.
- Conforms to UL® 1069 and CSA C22.2 No. 205R Standards, as tested by ETL. Durability tested to still function after being dropped 4 feet onto concrete 100 times, and withstanding static discharge in excess of 4,000 volts.
- Every Chair-Check II is assembled in the USA and 100% functionally tested before shipment. Two-year limited warranty.

Distributed by:

800-523-7956 | Fax 402-475-4023 | [www.bedcheck.com](http://www.bedcheck.com)

**STANLEY**  
Healthcare

# Chair-Check® II Models 72010 & 72011



## PURPOSE

The Chair-Check II control unit is intended to be used in conjunction with a Bed-Check® Sensormat® pad as a system for the monitoring of fall risk patients or residents in a bed, chair, wheelchair, or on a toilet. Chair-Check II provides an audible alarm to alert the caregiver when the person attempts to get up unassisted.

## COMPONENTS OF THE SYSTEM

The Chair-Check II and a pressure sensitive Sensormat pad (ordered separately) with a cord that connects to the control unit. Sensormat pads are available for bed, chair/wheelchair, and toilet. Chair-Check II has a convenient integral mounting clip. See Installation below for mounting options.

## POWER REQUIREMENTS

The Chair-Check II is powered by a common 9-volt alkaline battery, or optional rechargeable battery.

## INSTALLATION

The Chair-Check II is mounted on the bed headboard or frame, the chair/wheelchair back, or wherever convenient adjacent to the toilet, using optional wire holster or integral mounting clip. It should be inaccessible to the monitored person. The appropriate Sensormat pad is installed per its in-service instructions. See the User Guide for complete instructions.

## IN-SERVICE

Ask your professional Bed-Check Distributor about in-service training options.

## FEATURES AND OPERATION

The Chair-Check II is the most versatile control unit in Bed-Check's product line.

- Battery or AC operation (with rechargeable battery and AC adapter) allows use in portable or bed applications.
- 9V alkaline battery: battery life is "the gold standard" in the business, with low charge alert. Secured battery access.
- Controls for hold, tone, and delay are designed for easy changes by the caregiver, but not the patient.
- Caregiver can put the System on Hold while servicing the patient. Monitoring begins again automatically when the patient's weight is back on the Sensormat pad.
- Multiple tones allow Chair-Check II to be distinguished from other alarms or other Chair-Check II monitors.
- Delay is 0 seconds for most seated applications, since the patient is postured for quick stand up. 1, 3, or 5 seconds may be set for other circumstances, such as bed monitoring.
- Utilizes disposable, pressure-sensitive Sensormat pads available for chair, bed or toilet monitoring. Not intended for use with other manufacturers' pads.
- Non-latching alarm action. The alarm ceases and monitoring automatically resumes when weight is again sensed, reducing caregiver intervention compared to string-activated alarms.
- Integral stainless steel clip (see also Options below).

## OPTIONS

- Nickel-metal hydride rechargeable battery allows recharging >1,000 times. (Model 0450-013)
- AC adapter/battery charger (requires rechargeable battery) allows system use while charging the battery. (Model 0707-836)
- Holsters: Wire basket holster for easier mounting on chairs and headboards, with a 1" (Model 72042) or 2" (Model 72043) depth hanger.



800-523-7956 | Fax 402-475-4023 | [www.bedcheck.com](http://www.bedcheck.com)

